

# Adnuo® PD Meter

## Quick Start Guide

Measuring PD is easy. Follow these easy steps to obtain an accurate measurement:

1. Turn the device on. Flip up the metal forehead support (F8, see drawing on back).
2. The person whose PD is being measured will place their face against the PD meter. Make sure the forehead rests comfortably against the forehead support, and the nose support is gently resting on the nose. Keep the PD meter horizontal.



3. The observer will flip the monocular covering switch (B2, see drawing on back) to cover one of the eyes; you will measure one eye at a time.
4. Have the person being measured stare ahead at the center of the green target. Try not to blink.
5. The observer will move the black vertical line for each eye (fig. 1a) directly over the reflecting spot (white dot) in the pupil (fig. 1b) using the sliders on top of the device.



Figure 1a: Vertical lines are not yet moved.


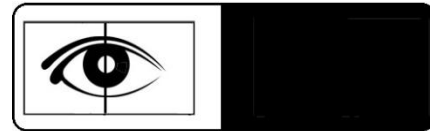



Figure 1b: Vertical lines are placed directly over pupils of the person being measured.

6. Flip the monocular covering switch to cover the unmeasured eye, and repeat steps 4 and 5.
7. Read the information off the display window. The number below "PD" is the distance PD for the person being measured. The numbers below "Right" and "Left" are the values for a person's monocular PD (see figure 2). Write down and save all 3 values.
8. In order to ensure accurate measurements, you may want to repeat these steps 2 or 3 times. When finished, take the average of each value. You are now ready to order your glasses online!

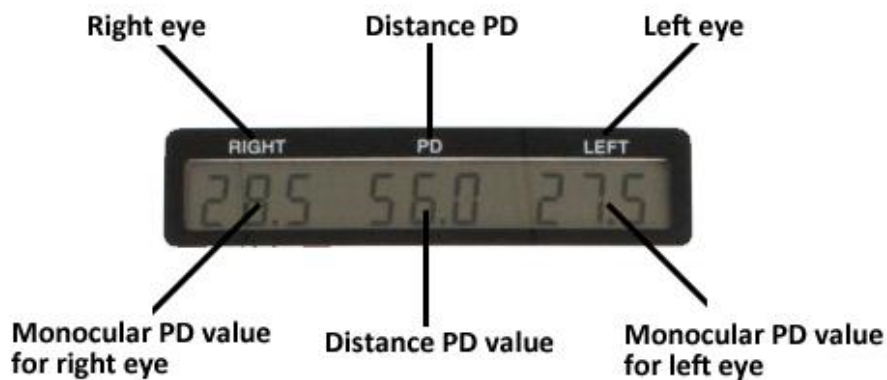

## PD Meter Parts

### Front:

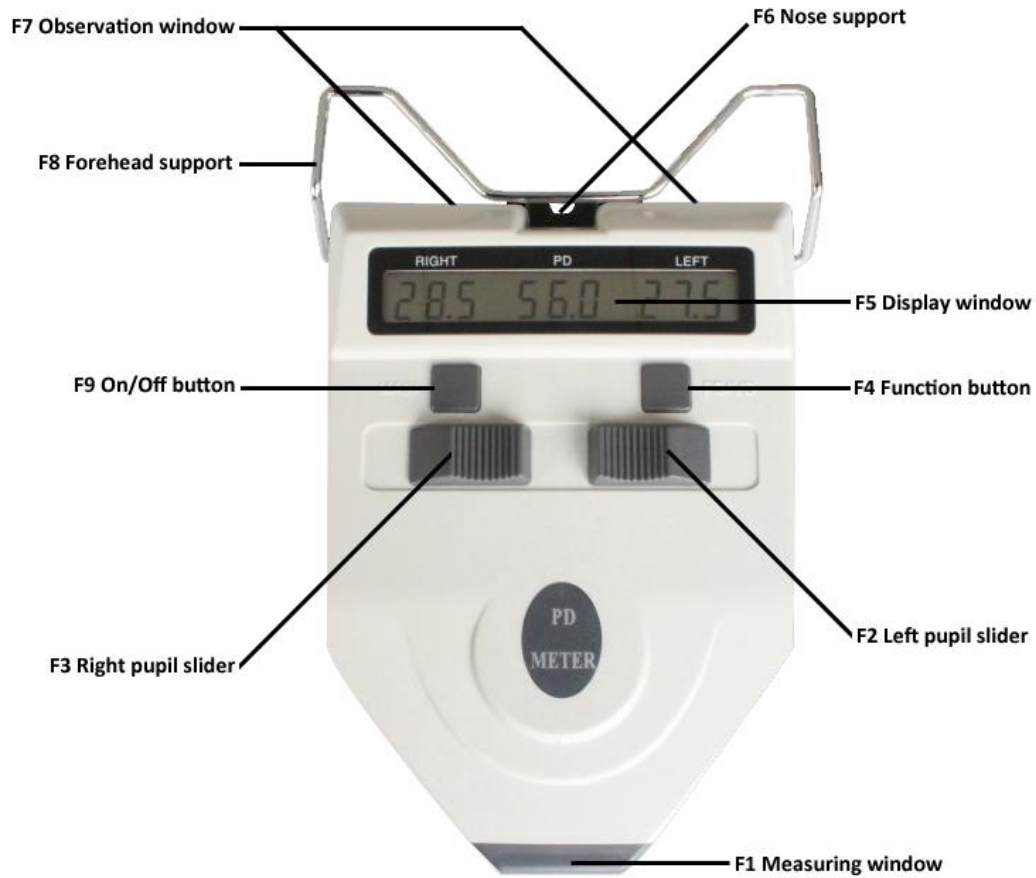

### Back:



See instruction booklet for detailed information.